


INTROCAN SAFETY 3

TIPS FOR SUCCESSFUL CANNULATION



FLATTEN THE WINGS

Flatten the wings prior to insertion. This will ensure optimal securement of the IV Device. Remove protective cap.

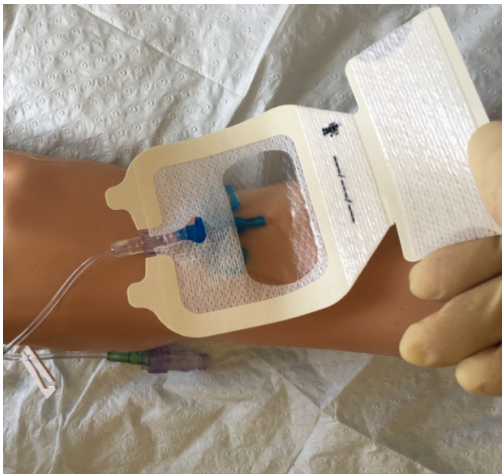

INSERTION

Hold the cannula by the flashback chamber (not by the wings) to puncture the identified site. Once initial flashback is achieved, wait for second flashback. Stabilise device. Remove needle.



SECUREMENT

Position the dressing with the notch lined up with the push off plate and press firmly into place. Peel away the frame and place securement strip over the catheter hub ensuring insertion site is visible. Label with date of insertion.